

Getting the most from your new iWOW®






Congratulations on choosing the iWOW® for your new ePR project. This guide will cover the various features and functions and help users get the most from the system.

Components to your new ePR cart:

- iWOW® ePR
- MT700 medical monitor
- MT202 hotswap battery pack
- MT250 6 bay charge station
- MT103 washable keyboard
- MT105 waterproof mouse

Prior to powering up

To power on your new iWOW® ensure 3 x batteries are installed in the rear of the unit (battery door will only close if batteries are inserted in correct orientation which is yellow label facing out).

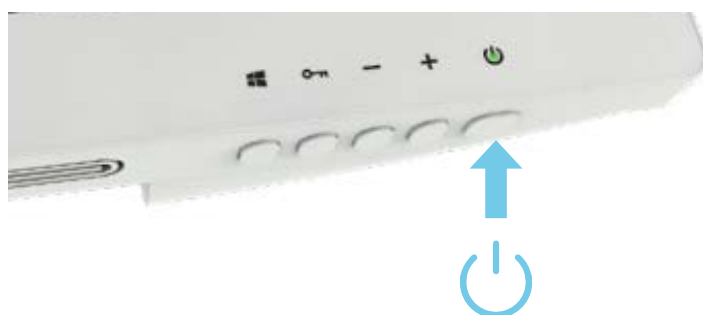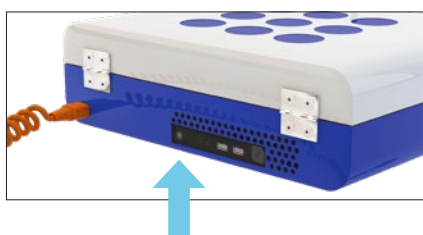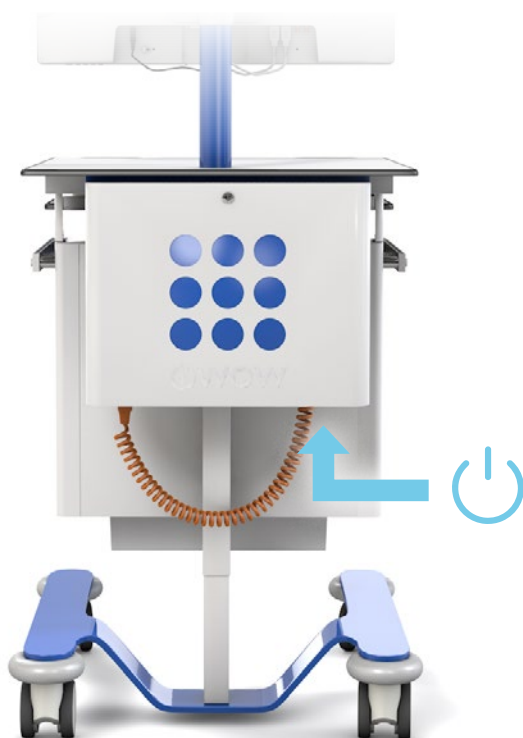

Switching on

First time powering on or if all batteries have previously been removed. Press the power on button (bottom right of screen bezel). The power indicator will light up orange. The screen is now powered and the PC needs turning on. To turn on the PC reach under the IT pod on the rear and press power button shown by arrow in green.

When PC is powered on screen indicator will change from orange to blue, Windows will load and you will be able to sign in to the system.

Pull-out keyboard/mouse tray


The keyboard and mouse tray pulls out and have negative tilt to allow the most comfortable use. Simply pull out and lift to change angle.

The palm rest for the keyboard can be slid to either side allowing the mouse to be used on either left or right hand side. To change side simply push it to the left or right.



Electric height adjustment

Cart height can be changed with the electric height button. Simply lift button up to raise height or push down to lower height. The button will display the height in centimetres allowing users to choose a consistent height.

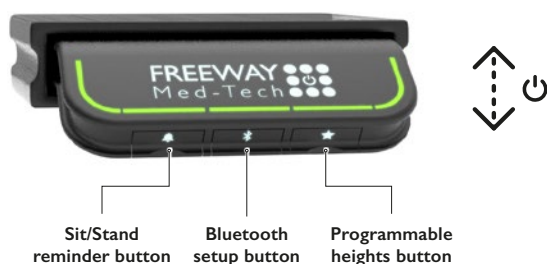

Digital height display - the up/down lever shows the floor-to-worktop distance when raising or lowering the cart (in cm).



Standard Height Adjustment

Standard equipment on iWOW® and iWOW® Keyboard, the lever-operated gas lift enables quick and easy height adjustment. All the heavy lifting is done for you on the way up and just a gentle amount of pressure is needed to lower the worksurface.



The monitor's built-in reading light

The MT700 monitor is equipped with a reading light for night rounds.

To turn on press the button located on the underside just to the right of middle. To change brightness press again and to turn off press and hold.

On-cart charging

Charging of the iWOW® can be achieved by plugging the cart into a wall socket.

All 3 batteries will charge at the same time. Approx time to charge is 6hrs.

Battery charging in progress is shown by flashing blue LED indicator top right of screen bezel. When flashing stops and indicator is steady, blue charging is complete.



Charging using the 6-bay charge station

Alternately batteries can be charged separate from the screen. This allows 24/7 operation by simply swapping batteries for fully charged ones at the start of a shift.

Batteries can be swapped without powering system off. Simply swap one battery at a time. Open battery door, remove battery by pulling upwards with fabric tab, replace with fully charged battery and close door. Note if all 3 batteries are removed at same time system will shut down and data may be lost.



Monitor battery status indicators

Battery status indicators on the screen bezel will allow you to ascertain when charging or swapping of batteries is required. Best practice is to charge or swap batteries when indicator changes to orange.

What do the LEDs mean?

- Blue indicates the battery is 25% to 100% charged
- Blinking blue indicates the battery is charging
- Orange indicates that the battery is between 11% to 25%
- Blinking Orange indicates that the battery is below 10%




The 6-bay charge station

When using MT250 charge station the indicator LED will show orange if battery is charging, blue if fully charged. Batteries can be left in charge station and will not overcharge.

Charge time is approx. 4 hrs

Charge station should be kept away from sources of heat (radiator, heaters, window cill in direct sunlight etc).

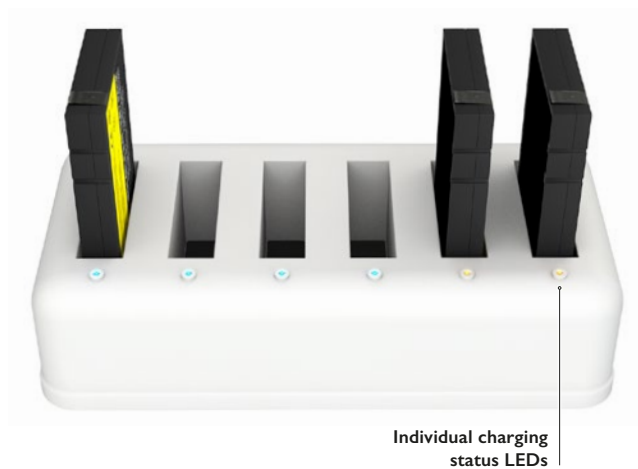

Electric lift - charging

If iWOW[®] is being used solely by hotswapping batteries and not being plugged into wall socket to charge, please note the battery for the electric lift will need charging approx. every 30 days.

The lift button will show orange or red status if charging is required.

General cleaning information

Cleaning of the iWOW[®] cart, screen, keyboard and mouse can be achieved with most standard hospital cleaners.

Whilst the screen is IP65 sealed (front only) it is advised to use either a wipe or apply cleaning fluid to a cloth before applying to screen.

Remember to remove any excess residue from the screen with a paper towel. Build up of residue may cause erratic touch screen function.




■ IP65 FRONT PANEL - MT700-24T

An IP65 front bezel. Infection control made easy.

The front of our medical-grade monitors are rated IP65 for dust and moisture ingress. Spray-on cleaners* and anti-bacterial cleaning wipes* can be used without worry of damage.

*Suitable cleaning chemicals only. See website for more info.

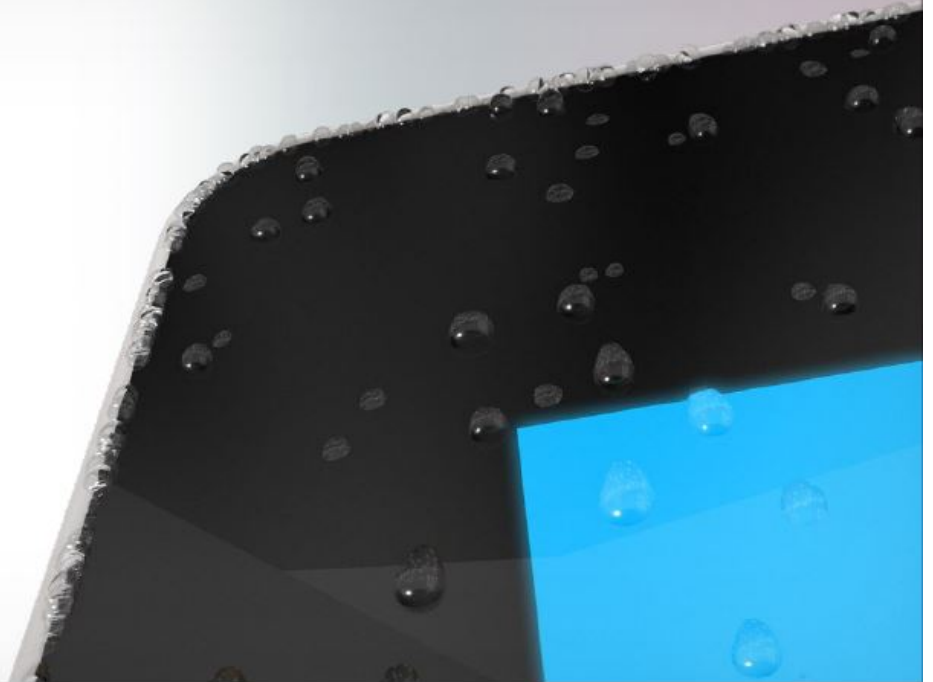

More detailed guides on operation of the cart, screen, batteries as well as best practices for cleaning and storage can be found on our website under our knowledge base section:
<https://www.freewaymedtech.com/services/support/technical-knowledge-base>