

BASIC OPERATION GUIDE

MT600-24T

Mains Powered Medical-Grade Monitor



Package contents

- I MT600-24T Mains Powered 24in Touch Screen Monitor
- I Medical Grade AC-DC power adapter with orange curly power cord
- I Basic Operation Guide
- I Cable pack

INTRODUCTION

Thank you for purchasing Freeway Med-Tech's Medical Grade Monitor. With a 24-inch display, the Medical Grade Monitor offers optimal combinations of performance and power savings. With fully-integrated point-of-care modules within an elegant, space-saving design, the Medical Grade Monitor is the optimum solution to enhance workflow and service, while paired with NHS Hospital Standard MFF PC, or thin clients.

Please take a few moments to review the contents of this document to ensure that the setup and startup proceed smoothly. The Monitor is ready for use, out of the box, in its default configuration when powered by the power source provided. The following documentation offers guidance on the hardware elements and features of the monitor. Please refer to your device provider for information pertaining to the software operating system or software applications.

The MT600-24T supports VESA standard (100x100mm) mounting brackets and arms. Attach the bracket or the arm to the back of MT600-24T, then use screws (4 x M4x12mm) to fix the VESA bracket or arm.

PRECAUTIONS

- Always exercise care when operating and handling the Medical-Grade Monitor.
- Never disassemble any portion of the enclosure, as this will void any product warranty on the Monitor.
- Do not use any AC/DC adapter other than the one provided with the device or a replacement acquired from the manufacturer.
- In the unlikely event that smoke, abnormal noise or strange odour is present, immediately power down the Monitor and disconnect all power sources. Please report the problem to your device provider immediately.

BASIC FEATURES

The Medical-Grade Monitor integrates a bright 24" display with a high performance system and USB ports for a comprehensive point-of-care solution.

Powering ON and OFF

Please use the AC-DC adapter with the Medical-Grade Monitor for the power supply. To activate the Medical-Grade Monitor, push and quickly release the Power Button and the display will come on in a few seconds.

CLEANING THE MT600-24T

The screen

- A screen cleaning solution that is alcohol-free and non-abrasive can be used to clean the touch screen.
- Cleaning with a micro-fiber cloth is recommended.
- Please spread the solution onto the cloth and then clean the touch screen.

The anti-microbial enclosure

- Use a soft/non-abrasive cloth moistened with water to clean the enclosure.
- If using a cleaner, an alcohol-free and oxide-free cleaning liquid is recommended.
- To prevent scratching the anti-microbial coating, please wipe gently.


I/O PORTS

The MT600-24T Monitor has a comprehensive set of I/O ports. The following ports are located along the lower rear edge of the unit.

FRONT




REAR

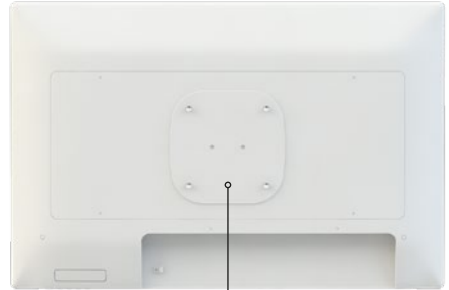

100x100 VESA mounting

A	DC-in	E	USB Type-B input
B	USB Type-B for touch screen connection	F	Power button
C	USB 3.0	G	Function buttons
D	HDMI (1.4a)	H	Stereo speaker

*The monitor's enclosure may look different to production models.

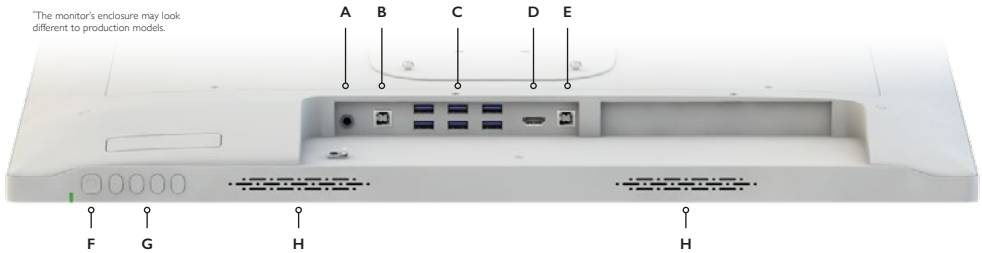

MEDICAL – GENERAL MEDICAL EQUIPMENT
AS TO ELECTRICAL SHOCK, FIRE AND MECHANICAL
HAZARDS ONLY IN ACCORDANCE WITH ANSI/AAMI
ES60601-1 (2005) + AMD (2012) & "CAN/CSA-C22.2
No. 6060-1 (2008) + (2014)

