


How to clean the Freeway Med-Tech MT600-24T Medical Monitor for infection control

Step 1. Freeway Med-Tech recommend the use of healthcare disposable alcohol based anti-bacterial wipes only on the electrical equipment (approx. 70% alcohol) – such as commonly used Clinell Green Wipes.

Step 2. It is recommended wiping from the front of the screen first in a top to bottom motion and then finish across the bottom (the front panel is IP65 Rated). This methodology is designed to minimise the amount of streaking on the screen and reduce excess moisture collecting on the lower front screen bezel, reducing the risk of moisture entering via the lower I/O, remembering to clean the lower OSD keys in the lower right hand corner. After the front Screen has been cleaned, dispose of this wipe.

Step 3. With a fresh Wipe, clean the rear of the AIO PC (the rear panel is IPX2 rated). To clean the screen of AIO PC for general house keeping - use either a microfibre cloth with an alcohol based cleaning spray (approx. 70% alcohol) or disposable alcohol based cloth.

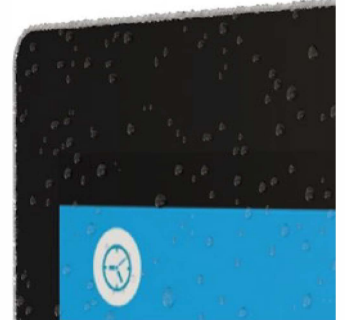

The front panel is IP65 Rated

How often can I clean my device?

You can clean your medical grade cart and Monitor as often as required. All health care institutions will have a recommended minimum frequency for routine cleaning of various items within their facility. This may be weekly, daily or after each use depending on the risk involved. Please refer to the below for all safe use chemicals.

Suitable Cleaning Chemicals

Given the vast range of cleaning products on the market, it is not possible to list all acceptable brands. We can however, confirm that our products are compatible with cleaning products containing the following chemicals or are from popular cleaning brands. Should you require confirmation of suitable products that are not listed, please contact us with the relevant product data sheet and we will confirm suitability with our in-house specialist.

Suitable cleaners/brands:

Cleaner	Brand
Viraguard®	Veridien
Sani-Cloth® HB	PDI
Sani-Cloth® PLUS	PDI
Super Sani-Cloth®	PDI
CaviWipes™	Metrex
Super HDQ®L 10	Spartan
Cloro-Wipe Towelette	Surgipath
Healthcare Hydrogen Peroxide Wipes	Clorox Healthcare®
Virex®	Johnson
70% Isopropyl Alcohol	Multiple suppliers
Alcohol Prep Pads	PDI, others

Suitable generic chemicals:

- Alcohol (Ethyl or Isopropyl) Solutions*
- Alcohol Wipes (Ethyl or Isopropyl)*
- Hydrogen Peroxide Solutions*

Cleaning procedure for nylon modified polyester coated surfaces:

- Painted surfaces should be cleaned using a soft damp non abrasive cloth and polished dry
- Continual exposure to wet conditions is likely to result in rusting/or the paint lifting.
ALWAYS DRY THE SURFACE AFTER CLEANING
- Continual use of cleaning products may result in a dulling of the surface.
- DO NOT use solvents, bleach, abrasives, synthetic detergents, wax polishes or aerosol sprays unless otherwise listed in the above list.

⚠ *Solutions must ALWAYS be diluted in accordance the with manufacturer's instructions for use on medical products/furniture.
*The application of the deaning solution MUST be in accordance the with manufacturer's instructions for use on medical products/furniture.